

Introducing

Pelican Elements

a complete guide to accessories



 [pelicanhealthcareltd](https://www.facebook.com/pelicanhealthcareltd)

 [@PelicanHealth](https://twitter.com/PelicanHealth)

[pelicanhealthcare.co.uk](https://www.pelicanhealthcare.co.uk)

pelican
HEALTHCARE

A close-up photograph of a man with short brown hair and a beard, wearing dark sunglasses, kissing a woman on the cheek. The woman has long reddish-brown hair and is wearing glasses and a denim jacket. The background is a blurred outdoor setting with buildings.

Pelican Elements:

Introduction

The Pelican Elements range is a selection of accessory products from Pelican Healthcare. These products are designed to solve a wide range of problems which you may encounter when living with a stoma. Each product has been specifically designed to benefit you.

To request complimentary samples and to find out more about the complete product portfolio available from Pelican, call us on **0800 318 282**, or visit our website **www.pelicanhealthcare.co.uk**.

- Deo-Mint
- Protect Plus
- Perform
- Release
- Contour Flange Extenders
- Select Minuet
- Ultra Barrier Cream
- Support



Added Vitamin E




Spearmint Fragrance



Available to Sample

**ALCOHOL
FREE**

Alcohol Free



"I have been using Deo-Mint for quite some time now, and it is the best product I have tried in the past 8 years for eliminating odour!"

E. Payne



Deo-Mint

Pouch Deodorant

Rather than masking odour, **Deo-Mint** eliminates and neutralises, leaving a refreshing spearmint fragrance. Available in a handy 50ml bottle, Deo-Mint can be sprayed inside the pouch during changes, and around the room.

- Refreshing spearmint fragrance
- 100% active liquid in every bottle



Code:

130770

Description:

Deo-Mint Deodorant

Pack Size:

50ml bottle



ALCOHOL
FREE



Protect Plus

Non-Sting Barrier Film

Protect Plus Non-Sting Barrier Film is designed to protect your skin from medical adhesives, enzymes and harmful body waste. The formula creates a barrier on the skin, which aids the adhesion and removal of pouches.

With the benefit of added Vitamin E, Protect Plus will continue to nourish and moisturise the skin.

Once applied, Protect Plus dries within seconds, making it easy and convenient to use. The spray also features 'bag-in-can' technology, which allows the user to spray in every direction, even upside down so it is easier to spray those 'hard-to-reach' areas.

- Creates a protective barrier on the skin
- Added Vitamin E
- No alcohol
- Refreshing spearmint fragrance

"I find the wipes easy to carry about and there is no problem of them drying out."

J.Holder



Code:

Description:

Pack Size:

130260

Protect Plus Barrier Film (Wipe)

30 wipes

130261

Protect Plus Barrier Film (Aerosol)


50ml aerosol

Perform

Solidifying Agent

Perform offers complete pouch control, taking away the worry of embarrassing leaks and pouch noise, by absorbing excess fluid and turning the stoma output into a gel. Designed for ileostomists, the clear sachet contains a highly absorbent powder, which upon contact with fluid, dissolves and solidifies the output, allowing controlled emptying and reducing any odour. Also during periods of physical activity, Perform will help to minimise the movement of output inside the pouch.

How To Use

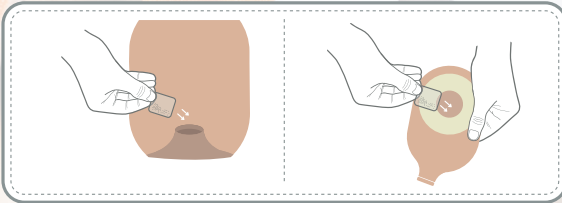

"I always use Pelican Perform. They are brilliant and give me lots of confidence."

J.Holder



Code:

130160

Description:

Perform Solidifying Agent

Pack Size:

60 sachets



"I love how there is no
chill on application."

M. Evans



Release

Adhesive Remover


Release is an effective non-sting adhesive remover, which leaves a refreshing spearmint fragrance when used.

Release allows for easy removal of the pouch, removing stubborn adhesive residue on the skin. With the benefit of added Vitamin E, Release will continue to nourish and moisturise the skin with every use.

The spray also features 'bag-in-can' technology, which allows the user to spray in every direction, even upside down so it is easier to spray those 'hard-to-reach' areas.

- **Added Vitamin E**
- **Refreshing spearmint fragrance**
- **Available as a wipe and aerosol**

How To Use

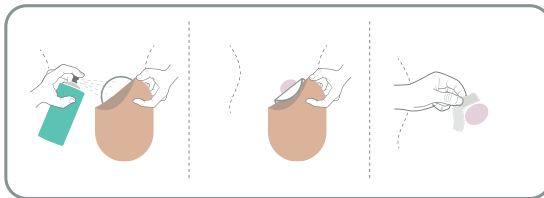

Code:

Description:

Pack Size:

130360

Release Adhesive Remover (Wipe)

30 wipes

130460

Release Adhesive Remover (Aerosol)

50ml aerosol




Select Minuet

Stoma Caps

For sporting activities, intimate moments and those who irrigate, **Select Minuet** Stoma Caps are small and discreet, allowing the user to experience the ultimate discretion. With an inner-cushioned layer and a waterproof filter, Select Minuet is also extremely comfortable, allowing the user to have greater security and confidence during the moments which matter.

- **Ideal for those who irrigate**
- **Suitable for ileostomists when bathing**
- **Available in 2 sizes**

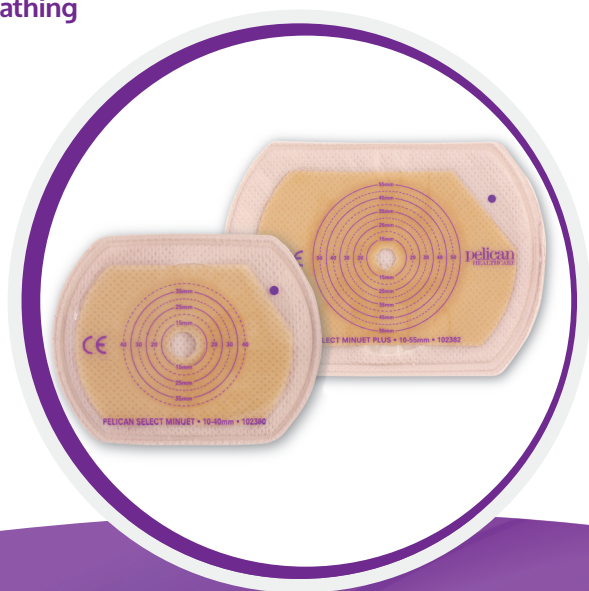

Code:	Description:	Size:	Pack Size:
102380	Select Minuet	10-40mm	30
102382	Select Minuet Plus	10-55mm	30

Ultra Barrier Cream

Water-Based

Pelican **Ultra Barrier Cream** is designed to protect and soothe the skin around your stoma, creating a safe, effective barrier against harmful waste or fluids.

Water-based for optimum absorption into the skin, just a small amount of ultra barrier cream is needed to protect peristomal skin.

The cream is also PH balanced, and so is ideal for use on sore and broken skin.

- **Designed to protect peristomal skin**
- **PH Balanced**

"I use your barrier cream and I just want to say thank you – it's a lifesaver."

J.Toombs



Code:


130105

Description:

Ultra Barrier Cream

Pack Size:

50g tube



**"When I wear the Support
Waistband I cannot feel the
weight of my pouch at all. It
is so supportive and figure
smoothing. I can't imagine
my life without it, so much
so, that I have 3."**

Pelican Support user

Support

Stoma Waistband

Pelican **Support** is a seamless support garment manufactured from a soft and breathable fabric that not only offers extra security, but also provides a smoother body shape for the wearer.

Suitable for both men and women, Pelican Support provides lighter support for small hernias, and is ideal for sporting activities, intimate moments, or anytime the wearer is looking for a smoother body shape.

- Available in 3 sizes
- Choice of black, white or beige fabric
- Holds pouch securely in place



Description:	Hip Measurement:	White:	Beige:	Black:
Small / Medium	85-110cm	130560	130561	130566
Medium / Large	105-125cm	130562	130563	130567
Large / Extra Large	120-140cm	130564	130565	130568



Eakin Range:

Introduction

TG Eakin Limited, the sister company to Pelican Healthcare, is a medical device manufacturer, dedicated to the production of high-quality skin protection products for use in stoma and wound care and has been successfully protecting skin for over 30 years.

The most common problems facing ostomates are leakage and skin irritation which can occur when output is particularly watery, when there are dips and crevices around the stoma or from using a poorly fitted appliance. Such leakage can then cause irritation to the surrounding skin, making it red and sore, and the Eakin range of products are designed to solve all these common problems.

- Eakin Cohesive® Seals
- Which seal is right for me?
- Eakin Cohesive Paste
- Cohesive StomaWrap™
- Eakin Support Belt



Added Vitamin E



Spearmint Fragrance



Available to Sample

ALCOHOL
FREE

Alcohol Free



Eakin Cohesive®

Seals

Cohesive® Seals and **SLIMS®** are fitted in position around the base of your stoma before applying your pouch, providing a soft and mouldable leak-proof seal.

Output from your stoma can irritate the surrounding skin, causing it to become red and sore. Eakin Cohesive® helps prevent waste from coming into contact with the skin, minimising any potential skin problems.

Cohesive® Seals and SLIMS® also allow you to keep your pouch on for longer and provide additional security.

"I use the Cohesive SLIMS® and they are superb. No skin reaction at all and they really last."

R.E. Harris



Code:	Description:	Diameter:	Thickness:	Pack Size:
839005	Cohesive SLIMS®	48mm	3.0mm	30
839002	Small Cohesive® Seals	48mm	4.2mm	30
839001	Large Cohesive® Seals	98mm	2.1mm	10



Which seal is right for me?

Cohesive SLIMS®

- For everyday skin protection and security
- For those who change their pouch frequently
- For mild skin irritation
- For use with pre-cut pouches to create an exact fit
- Ideal for use with convex pouches



Small Cohesive® Seal

- To provide maximum protection
- For those with high fluid content
- To resolve leakage problems
- To extend pouch wear time



Large Cohesive® Seal

- Ideal for urostomates
- To cover a large area of peristomal skin
- Helps prevent irritation from pouch flange
- For use with deep convexity





Cohesive® Paste

Alcohol Free

Cohesive Paste is an alcohol free paste, which absorbs moisture and blocks biological and chemical irritants coming into contact with the skin.

- Alcohol Free
- Absorbs moisture – keeping it away from the skin
- Will not sting or dry out
- Reduces digestive enzyme activity, stopping them from breaking down the skin.



Code:

Description:

Pack Size:

839010

Cohesive Paste

60g Tube



Cohesive StomaWrap™

Seal

The unique shape of Cohesive StomaWrap™ makes this seal very quick and easy to fit. There is no need to warm, stretch or tear to the correct size. Simply wrap it around the stoma and overlap to create a secure seal.

- Ideal for large or oval shaped stomas
- For those with limited dexterity

"I found them very easy to use and appreciate the fact that I didn't have to tear them at all. I saw a massive improvement with my sore skin even after just the first day of wearing them."

StomaWrap user



Code:	Description:	Diameter:	Thickness:	Pack Size:
839006	Cohesive StomaWrap™	85mm	3.0mm	10

Eakin Support Belt

Maximum Support

Eakin Support Belts are specifically designed to provide full and complete abdominal support for prevention and treatment of parastomal hernias.

Eakin Support Belts are manufactured from a unique material call Isoflex, which has been developed for ostomates to provide complete abdominal support.

- Lightweight, breathable and washable fabric
- Neutral tone fabric and slim line fit
- Assessment and measurement for a Support Belt should be carried out by a stoma care nurse



Description:	Hip Measurement:	20cm Depth:	26cm Depth:
Small	75-90cm	839051	839061
Medium	90-105cm	839052	839062
Large	105-120cm	839053	839063
Extra Large	120-135cm	839054	839064



Contact Us:

To request complimentary samples and to find out more,
about any of the products from the Pelican Healthcare range...

Freephone 0800 318 282
pelicanhealthcare.co.uk

 [pelicanhealthcareltd](https://www.facebook.com/pelicanhealthcareltd)

 [@pelicanhealth](https://twitter.com/pelicanhealth)



Pelican Elements

a complete guide to accessories